# ZirCam

## ZirCam Products Manual

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This catalog designed to help dentists  
to choose the right product share with  
his patients .

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## Catalog products are divided into:

### First: E.max products include the following products:

- 116 E.max staining crown
- 117 E.max layering crown
- 118 E.max veneers
- 119 E.max Endo/inlay/Onlay crown

### Second: Full zircon products from ZirCam:

- 127\* ZirCam Z-one shade crown
- 127\*\* ZirCam Z-ML crown
- 127\*\*\* ZirCam Z-UTML veneers
- 173 ZirCam Z-STML crown
- 241 ZirCam Z-UTML onlay /crown

### Third: ZirCam modified zircon products:

- 126 ZirCam Z-core with 2 ceramic layers
- 201 ZirCam Z-core with E.max 4 layers
- 202 ZirCam Z-UTML veneers with ceramic layers
- 186 ZirCam Z-Maryland bridge

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## Guide instructions :

1. Read each product carefully.
2. Use comparison between products.
3. Understand the advantages and disadvantages of each product.

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### **What are Z ZirCam Products?**

ZirCam products express the quality of the selection of zircon raw materials with the quality of their manufacture through a specialized team.

When you choose one of the ZirCam products, you choose the best for your patients at all for the following reasons:

1- The raw materials for zirconium are carefully selected from the best manufacturers of zircon discs at all, until companies want to have their products approved by ZirCam because they know ZirCam's ability to show them in a way that reflects the actual value of those materials,

In return for the high experience and advanced systems owned by the work team at ZirCam.

2- The raw materials used in the manufacture of ZirCam are registered with the Saudi Food and Drug Authority, which means that the dentist choose the best healthy materials, which means patient safety for the use of these materials.

3- The multiplicity of ZirCam products provides the dentist with wide options and also allows the patient to achieve his own desires and needs.

4- The raw materials for ZirCam products are constantly updated, and the customer is informed of those raw materials used in th manufacture of ZirCam when he desires to.

5- Complete reliability when choosing ZirCam products for project owners and long-term investors in front of their patients, which makes them feel absolute confidence with their patients.

6- ZirCam means continuity of customer service and non-stop for any reason, due to the presence of multiple systems and skilled technicians, which ensures that the work is not disrupted technically or humanly.

7- ZirCam is a Saudi registered trademark wholly owned by the Saudi Dental Lab Tech,

their use by owners and dentists increases the confidence of the final consumer, which means an excellent marketing addition to the investor.

### **How is it manufactured?**

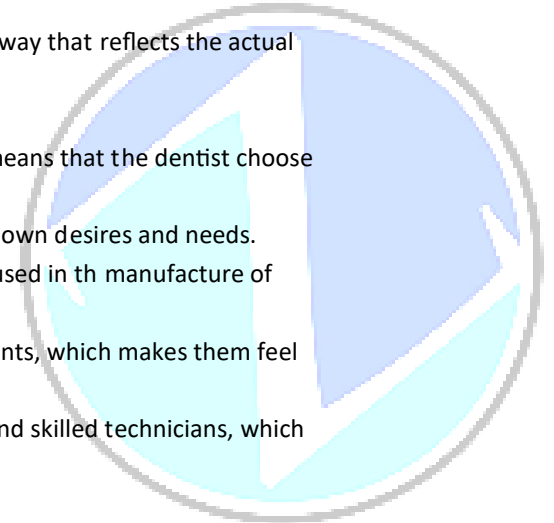
1- By selecting the raw materials after obtaining the specifications of the product to be in conformity with the standards of ZirCam.

2- Samples are checked before they are entered for manufacturing.

3- The product is made through manual or digital impressions , through the computer design programs through the most famous design software in the world.

4- It is also turned by integrated software advanced milling systems with high strength .

5- The final steps are through skilled technicians who knwos what dentists need.

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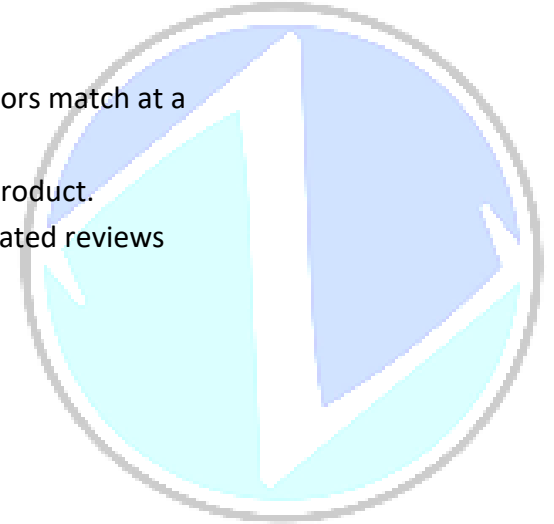
### **Criteria for selecting raw materials for ZirCam:**

- 1- The raw materials manufacturing company (country of origin) should be well-known in the manufacture of zircon discs and have a well-known trade name.
- 2- The zircon materials must be registered with the Saudi Food and Drug Authority.
- 3- Passing the color matching test that is carried out in the Saudi Dental lab tech, and that the colors match at a high rate, especially in difficult colors.
- 4- Passing the stage of experience with customers and their satisfaction with the manufactured product.
- 5- The selection of raw materials for zirconium discs is stopped immediately when there are repeated reviews about the product from customers.

### **Advantages of ZirCam products:**

- 1- Multiple products.
- 2- Variety of prices.
- 3- High-cost, medium- and low-cost products.
- 4- Multiple continuous offers.
- 5- Creating new products that meet of customers needed.

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## 116 E.max staining

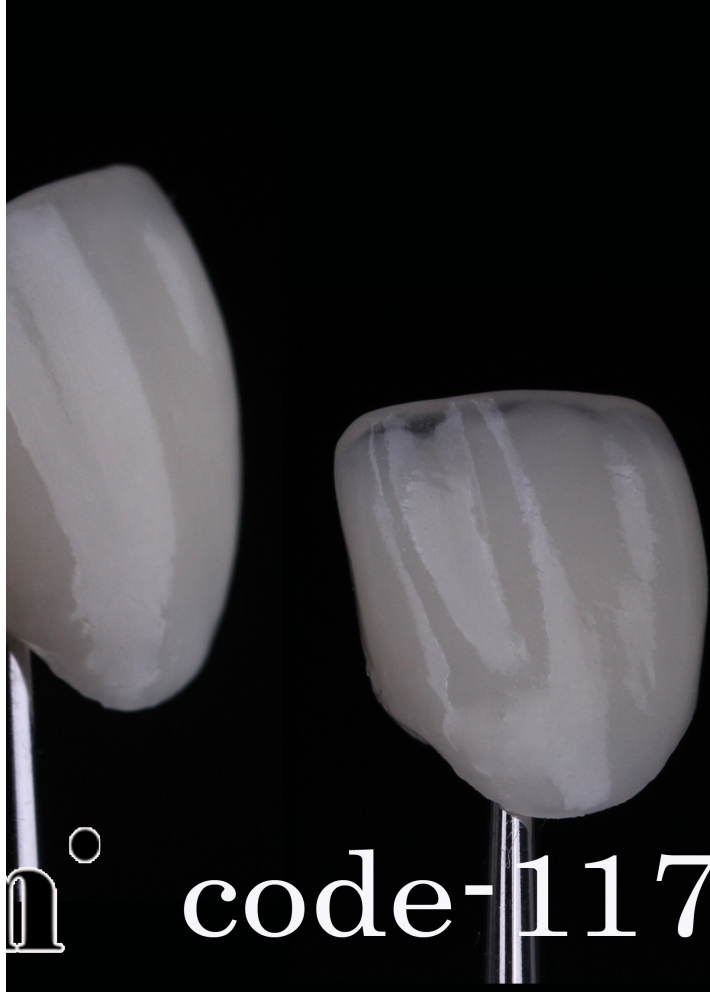
Come in multiple colors cubes ingots that meet the customer's need.

Crown for posterior teeth ,indo crown , anterior crown with conditions.

Adhesive strength, suitable against chewing forces

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## 117 E.max layering crown

**bright smile with a striking transparency to satisfy the customer's need to excel in appearing and feeling the vitality of youth.**

**Processed with CAD CAM systems, injection devices and furnaces , available in multiple colors.**

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**Super aesthetic. Attractive appearance .High adhesion**

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### 118 E.max veneers

**Bright smile with a striking transparency to satisfy the customer's need to excel in appearing and feeling the vitality of youth.**

**Choose of stars and celebrities to get a smile that makes him confident .VIP customers request**

**The product is completely Swiss made, processed with German CAD CAM systems, German injection devices and furnaces.**

**Available in light colors**

**Ceramic veneer for Anterior teeth up to No. 5 on each side of each jaw**

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**119 E.max Endo/inlay/Onlay crown**

**This products for restoration apart  
of tooth as a esthetic or functional**

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## **127\* ZirCam Z-one shade**

**Are you looking for strength? This product is designed for that.**

**Aesthetic medium, especially for dark colors. A good aesthetic for white color and ice whites, suitable for functional bridge use.**

**Suitable for people with low incomes and some special cases.**

**Anterior & posterior crown & bridge ,it can be made as a bridge for 3 missing teeth. In certain cases ,it can be used as a celebrity smile for white as crowns, not use for veneers.**

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**127\*\* ZirCam Z-ML crown**

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**Full jaw bridge.it is not recommended for use on missing teeth for more than 2 for posterior teeth or 4 for front teeth**

**Natural aesthetic , fine-looking. excellent adhesive strength**

**Choose this product and you will not regret**

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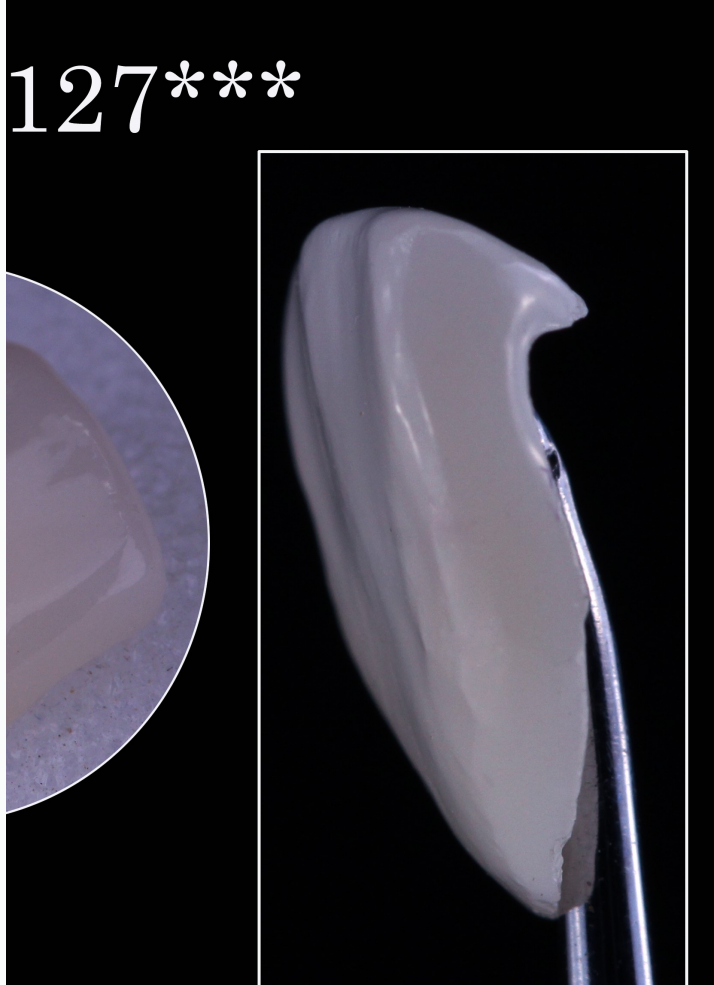
This product also has the ability to make crowns and veneers in the same condition without the color contrast

Natural aesthetic, Fine-looking ,  
excellent adhesive strength.

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It is not recommended for use on missing teeth for more than 2 for posterior teeth or 4 for front teeth

Natural aesthetic, fine-looking ,excellent adhesive strength.  
Celebrity smile.

A product that combines strength and aesthetic

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**It is not recommended for use on missing teeth for more than 2 for posterior teeth or 4 for front teeth**

**Natural aesthetic, fine-looking ,excellent adhesive strength.  
Celebrity smile.**

**A product that combines strength and aesthetic**

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### 126 ZirCam Z-core with 2 ceramic layers

The most common and requested product made of zirconium

Zircon materials are selected from the best international names and licensed by the Saudi FDA, coming in white or colored tablets. European porcelain materials (German,

Anterior crown & bridge. Posterior crown & bridge . full jaw bridge.it is not recommended for use on missing teeth for more than 2 for posterior teeth or 4 for front teeth

Natural aesthetic .Fine-looking . Excellent adhesive strength

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It is not recommended for use on missing teeth for more than 2 for posterior teeth or 4 for front teeth

High aesthetic .fine-looking excellent adhesive, strength.

It differs from product code 126 in that this product has more transparency because for our use of E.max porcelain by 4

**Foundation durability and aesthetic**

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**202 ZirCam Z-UTML veneers with ceramic**

Zircon shell with ceramic veneer for front teeth up to  
No. 5 on each side of each jaw

Choose stars and celebrities to get a smile that makes  
him confident.

VIP customers request.

**Have a distinct smile with zircon and clear porcelain**

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## 186 ZirCam Z-Maryland bridge

Used to solve the problem of losing one tooth from the front and the dentist or patient does not want to make permanent bridge or implant .

Very suitable in cases of quick cosmetic

Product s Code	المنتجات Products	Inla y	Posterior Crown	Anterior crown	Posterior Bridge 3 units	Long bridge	Anterior bridge 3 units	veneers	Pontic 3 units	Anterior Pontic 4 units	Endo crown
116	E.max staining crown		√								
117	E.max layering crown			√			√				
118	E.max veneers							√			
119	E.max Endo/inlay/Onlay crown										√
127*	ZirCam Z-one shade crown		√		√	*√			√		
127**	ZirCam Z-ML crown		√	√	√	√	√			√	
127***	ZirCam Z-UTML veneers							√			
173	ZirCam Z-STML crown		√	√	√	√	√			√	
241	ZirCam Z-UTML onlay /crown	√	√	√			√				√
126	ZirCam Z-core with 2 ceramic layers		√	√	√	√	√			√	
201	ZirCam Z-core with E.max 4 layers		√	√	√	√	√			√	
202	ZirCam Z-UTML veneers with ceramic layers							√			
186	ZirCam Z-Maryland bridge						√				

\*It means with white color

## Zircam is one of the trademarks owned by the Saudi Dental Lab Tech

### The goal of this guide

This guide was designed to meet the needs of the dentist so that he can increase his sales through the various products of zircon, and help him choose the appropriate product based on the clinical diagnosis.

### How to use this guide?

10 diverse zircon products that are designed across the cad cam, taking into account the quality of raw materials and workmanship, are designed to give the dentist the opportunity to choose multiple of these products.

The dentist should first see the ten products and understand them and understand the materials, properties of each product, as we in the Saudi Dental lab Tech put in the hands of the dentist all communication channels so that we help him in the most appropriate selection according to important factors:

First: Knowing the patient's desires and needs, this is very important in determining the type of work. Second: Knowing the patient's physical ability, Third: Classifying the patient's condition.

### **How is the patient's condition classified according to the clinical examination?**

Aesthetic compensation - functional compensation - functional cosmetic compensation - cosmetic functional compensation.

### **About this guide:**

This guide is the first version where updated versions will be made every 4 months or six months with the development of new products in it according to what is being created or what is being created from international companies.

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11- A patient wants to make a back bridge, but his problem is that he has 34, 35 and 36 missing, so what are the best materials for him?

In this case, the best choice is the metal covered with porcelain, and in the event the patient insists on zircon, the service is 127 \* is the best type of zircon for him, but this type is less aesthetic but the important thing is the functional aspect, in this case adequate preparation must be made for the age to create an occlusive distance of up to 1.5 mm.

12- A patient with anterior teeth 11, 12, 22, 21 is missing and wants to do a functional and cosmetic installation. What are the best products?

In this case, we recommend that you perform a canine preparation with No. 4 on both sides, and you can choose between the products: 126, 127 \*\*, and 173 in the event of a functional and aesthetic combination which is the highest cost or the product 127 \* as a functional and aesthetic aspect less and it is the lowest cost.

13- Is it possible to combine more than one product for one patient? In case yes we want an example?

Yes, more than one patient product can be combined according to its budget. For example, we have a patient who wants to make full compensation for the upper jaw crowns, so the front teeth can be made from the canine to the canine from product 127 \*\*\* and the background from product 173, 126 or 127 \*\* depending on the color of course and its availability. The higher the colors, the more choices you will have.

14- Can the patient attend the Saudi Dental Technology Laboratory to take color or ask about the type of materials?

Yes, with pleasure.

15- We hear that zircon is strong, so will it be broken?

Zircon has a strength of approximately 1,200 MPa, upon completion of manufacturing, in the product 127 \* only. The rest of the products decrease strength whenever the amount of zircon is less or modified hips from the factory to create transparency, so it is important to fully understand this. Products 127 \*\*\* The lowest strength of zircon is therefore strong. It is used only for crowns and lenses, and it is like Emax. Its strength after gluing is derived from the strength of the natural tooth to form with it one piece together. Product 126 has less strength, so the connections are designed in a way that resists the fracture, so we do not advise the doctor to use this product. For a bridge with 3 missing teeth, the doctor is told that the laboratory is not responsible in the event of a fracture.

16- Is there a guarantee on the products?

There are two types of warranty:

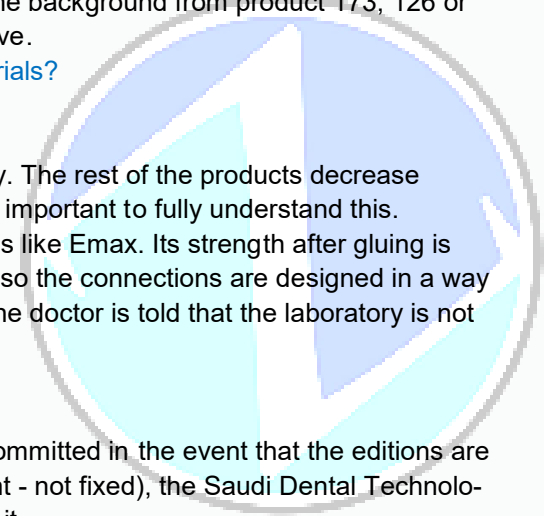
The first type before installation: Ensuring successful installation, the Saudi Dental Technology Laboratory is committed in the event that the editions are accepted to manufacture the product and if it is not installed for any reason (not revealed - large - color different - not fixed), the Saudi Dental Technology Laboratory is obligated to amend the installation or return Manufacturing after knowing the reasons to avoid it.

The second type of post-installation warranty: Most products in the event of acceptance of work - and there was no exception by us to the doctor for the possibility of fracture - has a two-year warranty, and the warranty includes fracture and discoloration only and does not include any other complications that may occur to the patient from gingivitis or allergic reactions due to which The doctor may break the bridge for re-treatment, as this is not guaranteed by the Saudi Dental Laboratory, as this is the prerogative of the doctor.

Products 202 and 127 \*\*\* The lenses and 118 are not included in the post-warranty warranty, but they are included in the warranty before installation, and the reason for not including the warranty after installation is due to:

A- The materials that the doctor uses to paste, this is due to the clinic with the patient if they wish to ensure the adhesive or not, so the Saudi Dental Technology Laboratory does not bear that.

B - The patient's behavior in preserving them, because the bad use exposes the lenses to break and fall, so the Saudi Dental Technology Laboratory does not bear that.

common questions

1- What does full zircon mean?

It is a compensation for one tooth or more of complete zircon without building a ceramics on it.

2- What is the difference between Product 116 and Product 117?

The difference is that the product is 116 fully pressurized porcelain and the 117th product is a compact porcelain foundation with a 4-layer E.max construction

3- What is the difference between product 117 and product 118?

The difference is that product 118 is only for ceramic veneers.

4- What is the difference between product 117 and product 126?

Product 117 is a compact ceramic foundation with a 4-layer Emax construction, while Product 126 is a zircon foundation with a two-layer ceramic construction.

5- What is the difference between product 126 and product 127\*\*\* ?

Product 126 is the basis of zircon with a two-layer ceramic construction, while product 127 \*\*\* is a fully transparent zircon, and also differs in the indications for use.

6- What is the difference between products 127 \*\*\* , 127 \*\* and 127\* ?

All of these products are classified within the complete zircon without building ceramics on them, but the product 127 \*\*\* is the most transparent and is used only for crowns and lenses while the product 127 \*\* is less transparent and used in bridges, while the product 127 \* is the strongest at all, but the product 127 \* comes Single color The products 127 \*\*\* and 127 \*\* come in gradient color with different transparency.

7- How does product 173 differ from other products in question 6?

This product is a highly transparent zircon , but it is slightly less than the 127 \*\*\* and this product differs from the 127 \*\*\* in that it is used in crowns, bridges, and jaws in its entirety.

8- What is the difference between product 127 \*\*\* and product 118?

The difference between product 127 \*\*\* and product 118 is that the former is made entirely of transparent zircon, while product 118 is made of compact porcelain based with the construction of 4 layers of EMAX porcelain.

9- What is the difference between products 127 \*\*\* , 118 and 202?

Product 202 differs from the products (127 \*\*\* and 118) described in Question 8 that product 202 consists of a zircon basis for product 127 \*\*\* with a porcelain construction on it transparent.

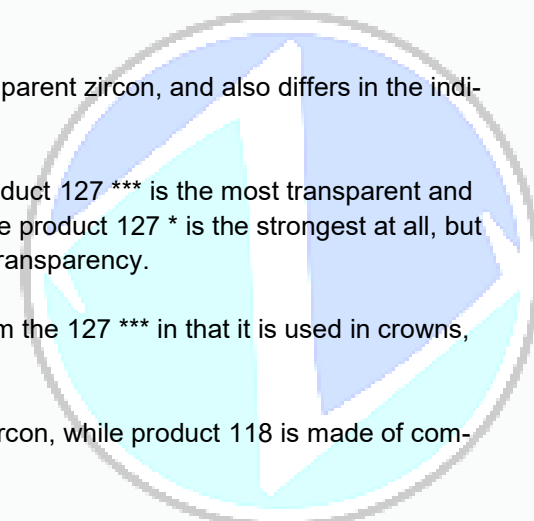
10- A patient wants to make a Hollywood smile and has pigment problems in his front teeth, so what should you advise him?

He has options for either the service lenses 118, but the doctor must have knowledge of how to choose the color and also the method of pasting through

the available colors for pasting so that he can make a match between the background colors of the natural teeth and this service is expensive compared to

others, or he has the service option 127 \*\*\* if he Afraid of the contrast of the color later, which is less expensive. He also has the option of 202 service,

which is medium cost. It depends on the patient's purchasing power .

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**Product code 127\*\*\* : Zi. trans(full) Anatomical (UTML)**

Classification : cosmetic  
For veneers & crowns

**Price :450**

**Product code 116 :All ceramic E. max pressing**

Classification : Functional  
Use for posterior crown

**Price : 450**

**Product code 127\* : full anatomical Zir .one shade**

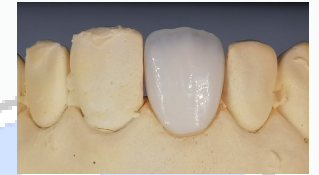
Classification : Functional  
For posterior crown & bridge

**Price : 300**

**Product code: 117 All ceramic E .max layering**

Classification : cosmetic . Functional  
Use for anterior & posterior crown

**Price : 500**

**Product code 173:Zir trans full anatomical STML**

Classification : cosmetic—Functional  
For crown& bridge full arch

**Price : 450**

**Product code 118 : E.max Veneers**

Classification ; cosmetic  
Hollywood smile

**Price ; 650**

**Product code 127\*\* :full Zir . low trans (ML)**

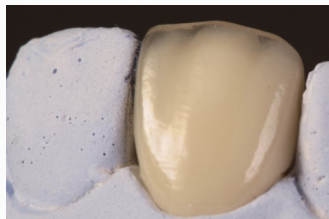
Classification : functional & cosmetic  
For crown & bridge

**Price : 350**

**Product code 126 :copping Zr with porcelain covering**

Classification : cosmetic—functional  
Most sales

**Price ; 315**

**Product code 201 :E.max Ceram on Zir core**


Classification : cosmetic  
For cosmetic & functional

**Price 400**

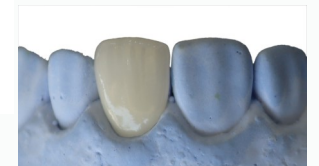**Product code202 ;Zircon veneer with high translucent ceramic**

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